MARATHON-III

Motor Specs

1.Rated voltage : 240V /50HZ 2.Power Input: 65W 3.Out voltage: DC 0-32V 4.Rated Speed: 0-50,000 rpm (Max)

Instructions

1. Avoid changing the rotational direction until the unit has stopped running. Changing the rotational direction too frequently will cause damage to the unit.

2.Please ensure the speed control knob is turned down (counterclockwise to '0') when finishing work. This will decelerate the rotation to a minimum before turning off.

3. Always use the rest after operating the machine and leave a bur or mandrel inside the chuck when not in use. This increases the life of the unit and prevents damage to the collet.

4. When adjusting the speed, DO NOT press the rotation switch. Please wait for the unit to stop and then the directional rotation can be safely changed.

Package Contents

1x Micromotor 1x Handpiece 1x Foot switch 1x Handpiece rest 1x User Manual(English) 1x Chuck tool 2x Replacement brushes

Warranty

Handpiece 1 year, micromotor 1 year, foot pedal 1 year. Warranty does not include misuse or abnormal wear of: bearing, carbon brushes, collet chuck, spindle & armature